KENMARK EYEWEAR

INSTRUCTIONS FOR USE

Sunglasses

Description

The Kenmark Sunglasses are considered Category 1 personal protective equipment since they protect users from damage to the eyes due to exposure to sunlight (when not directly observing the sun).

Kenmark Eyewear Sunglasses have the following features:

- Select models include polarized lenses, which enhance visibility by reducing glare from reflective surfaces thus providing eye protection and contributing to safety whilst outdoors. For example, when driving or working outdoors in wet or snowy conditions.
- Select models include ultraviolet light (UV) protection, which safeguards the eyes from harmful ultraviolet light radiation and helps prevent cataracts, macular degeneration, and other eye issues.

Intended purpose

Tinted sunglass lenses may be used to protect the eyes from bright light and radiation.

Intended users

Intended users are people who want to protect their eyes from damage to the eyes due to exposure to sunlight.

Contraindications

No contraindications have been identified.

Warnings

- The tinted sunglass lenses in this product are not suitable for direct viewing of the sun.
- The tinted sunglass lenses are not suitable for use in poor lighting conditions.

Precautions

- Do not use hot water during cleaning it can warp your frames.
- Do not use dish soap with additives like lotion.
- Do not use abrasive chemicals like acetone or ammonia it can strip off any lens coatings.
- Do not use paper towels as they may scratch your lenses.
- Do not fog your lenses with your breath before wiping it promotes the spread of germs.
- Do not use the dish soap method for wood frames the water will warp them!
- Do not use your shirt to dry your frames it can scratch your lenses.
- When not in use keep eyewear stored away in your case. A hard shell case is preferred!
- Don't forget to get the nooks & crannies! Nosepads and earpieces are commonly overlooked.
- Make sure your microfiber cloth is regularly cleaned. Your freshly cleaned eyewear is back to square one if you're using a dirty cloth to wipe it dry!

How to Use

The product is to be worn on the face, resting on the nose and ears. Your eyecare provider will prescribe suitable corrective lenses as needed.

Clean with mild detergent free from lotions and dyes and warm (not hot) water.

Do not use abrasive cloths when cleaning sunglasses.

Dry with clean microfiber cloth.

Transport and Storage

The frames are designed for everyday indoor and outdoor use in normal environmental conditions. The spectacle frames are intended for personal use in common daily activities such as reading, driving, and general day-to-day tasks. When not in use keep eyewear stored away in your case.

Instructions for Disposal

Ensure Sunglasses are disposed in accordance with local regulatory requirements.

The sunglasses are manufactured in accordance with ISO 12312-1:2022 for sunglasses and fashion spectacles.

Documents may be accessed at <https://www.kenmarkeyewear.com/>.

Symbols

	Indicates the medical device manufacturer. The symbol is accompanied by the name, registered trade name or registered trade mark of the manufacturer and the address of its registered place of business.
	Indicates the Authorized Representative in the European Union. This symbol is accompanied with the name or registered trade name of the Authorized Representative and the address of its registered place of business.
	Indicates the date when the medical device was manufactured. This symbol shall be accompanied by a date to indicate the date of manufacture. This shall be expressed in format YYYY-MM-DD in accordance with ISO 8601-1. The date shall be located adjacent to the symbol.
	Indicates the manufacturer's batch code so that the batch or lot can be identified. This symbol shall be accompanied by the manufacturer's batch code adjacent to the symbol.
	Indicates the manufacturer's catalogue number so that the medical device can be identified. This symbol shall be accompanied by the manufacturer's catalogue number adjacent to the symbol.
	Indicates the entity importing the medical device into the applicable market in scope. This symbol shall be accompanied by the name and address of the importing entity adjacent to the symbol.

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